



Fitting & Cut-Out

1. Adjust the frame
2. Place the frame on the patient
3. Dot the fitting height, 2 or 3mm above the lower lid, as in the illustration
4. Take a distance P.D. measurement
5. Place the dotted lens over the top of the segment outline shown in the right or left lens illustration
6. Minimum segment fitting height is 14mm
7. Confirm that the lens will cut out

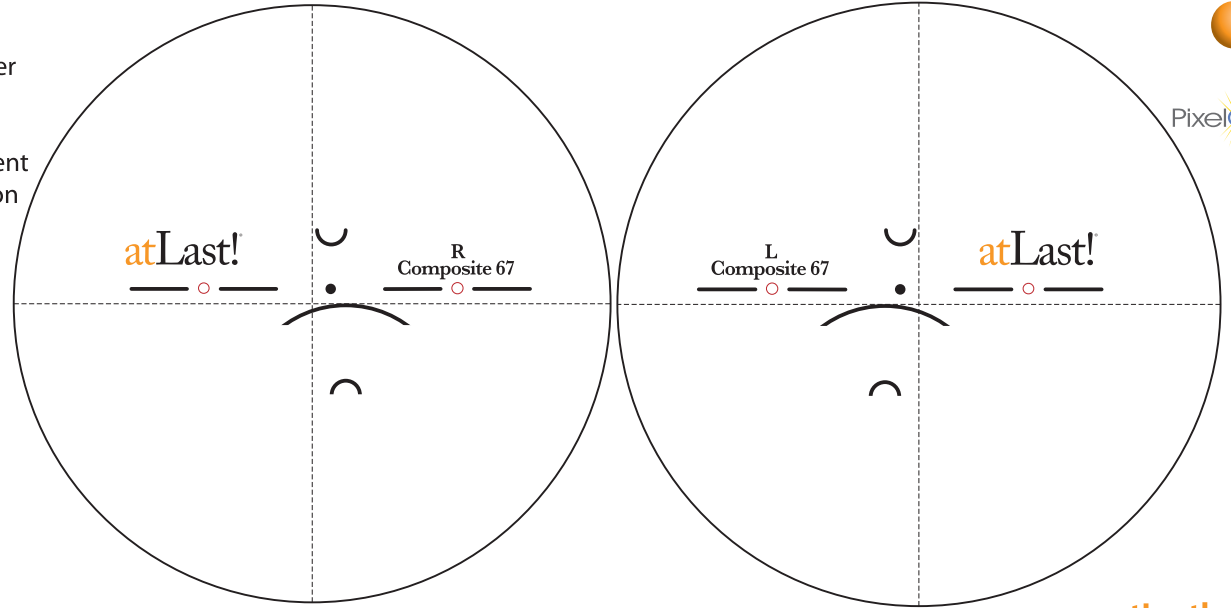

atLast

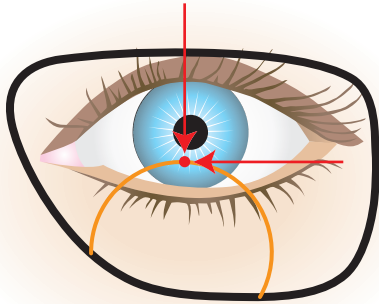

PixelOptics™

www.atLastlens.com

U.S. Patent No. 7,475,985. Other US and Foreign Patents Pending.

©2009 PixelOptics. All rights reserved. atLast!® is a registered trademark and PixelOptics™ is a trademark of PixelOptics Inc., Roanoke, VA.

PUB-000000009